




## Quick Start Guide


### FOOD PREP

#### 1 SELECT FOOD

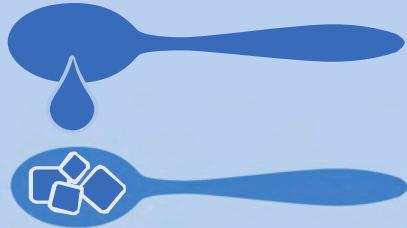


Liquids: soups, cereals, beverages  
Semi-solids: oatmeals, mashed potatoes, puddings  
Solids: fruits, vegetables, meats

#### 2 PREPARE FOOD

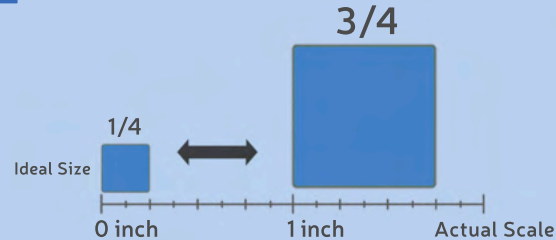

Obi works best when food is prepared within the range above.


#### 3 INSERT FOOD

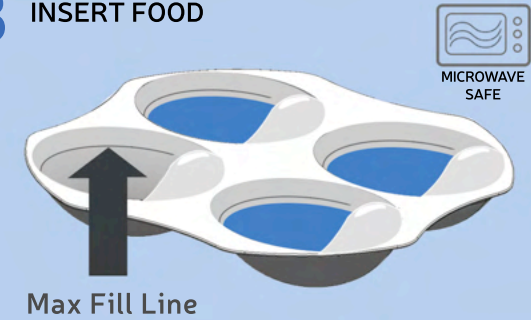

Fill bowls with food. Overfilling is not recommended.


### SETUP

#### 4 INSERT SWITCHES

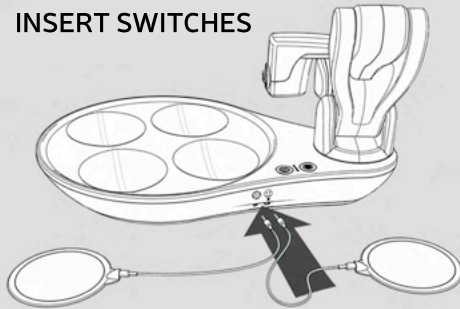


Position the device on a stationary, dry, horizontal surface.  
Move the Arm to the right, as shown.  
Insert the desired Accessibility Switches.

#### 5 INSERT PLATE & PLACEMAT

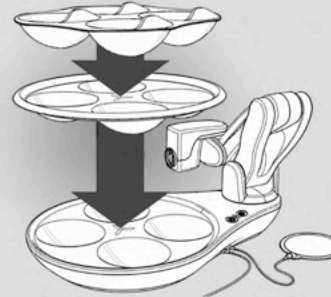

Clean Plate and Spoon before first use.  
Insert the colored Placemat, then place the Plate on top.


#### 6 ATTACH SPOON

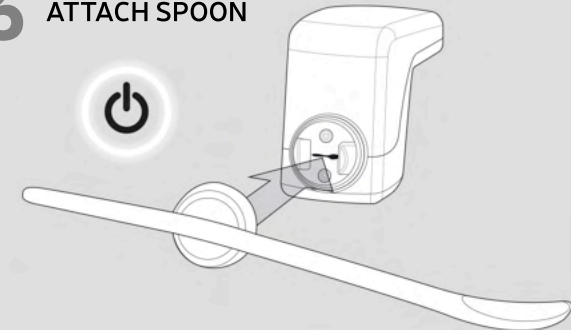

Attach the Spoon and press the Power Button.


### TEACH MODE

#### 7 ENTER TEACH MODE

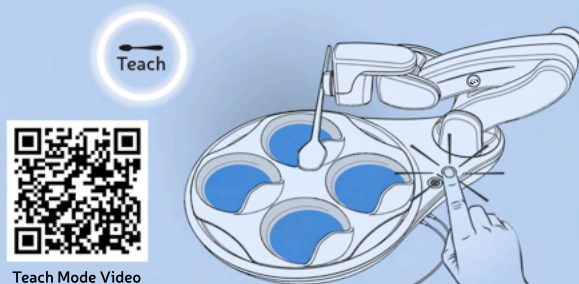


Position the user to be upright and stationary.  
Press the Teach Button.

#### 8 DETERMINE LOCATION

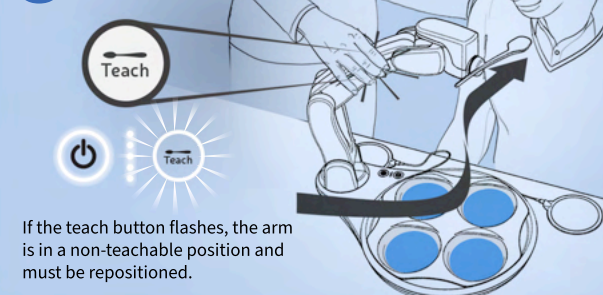

Grasp Arm with the thumb covering the Teach Icon. Lift and rotate to position the spoon in front of the user's mouth.  
**⚠ WARNING: CHOKING HAZARD.** Do not teach a delivery position inside of the mouth.

#### 9 EXIT TEACH MODE

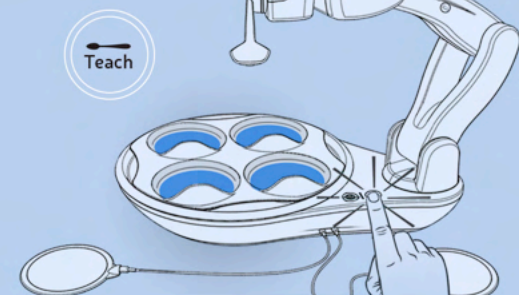


Press the Teach Button again to store the position.



## Quick Start Guide


### EATING

#### 1 CHOOSING FOOD

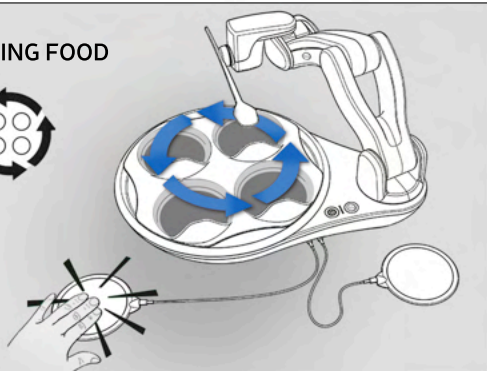


Touch the Choose Switch to move the Spoon one bowl counter clockwise.

#### 2 DELIVERING FOOD

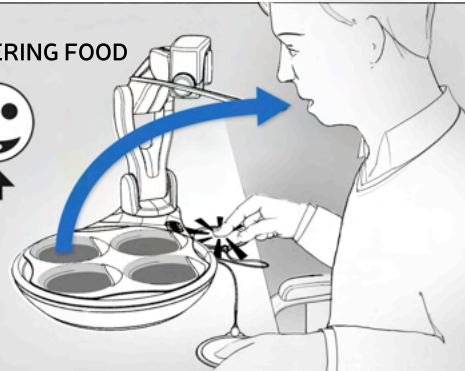


Touch the Deliver Switch to scoop and transport food to the delivery location.

#### 3 RETURNING ARM

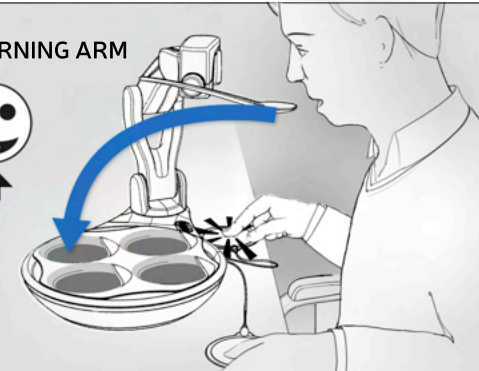


Touch the Deliver Switch to return the Spoon to the bowl.

### CLEANING

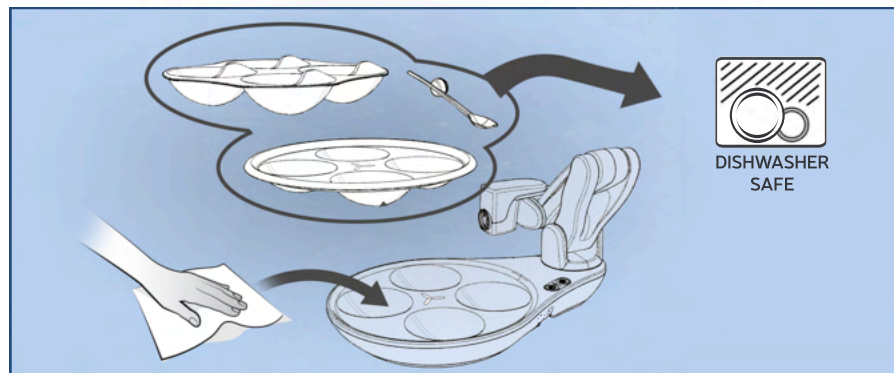


Clean the Plate, Placemat, and Spoon in a sink or dishwasher (top rack).  
Clean Obi's Base and Arm with a disinfectant wipe.

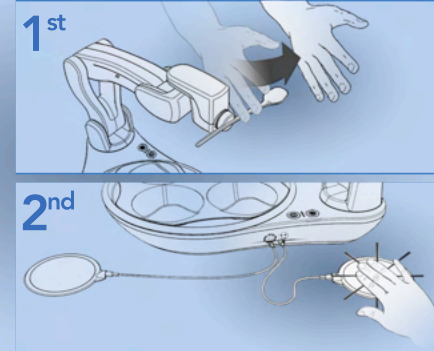

If the Power Button flashes, please remove the obstructing object. Once removed, touch either switch and the Arm will resume its motion.

### COLLISION

### CHARGING

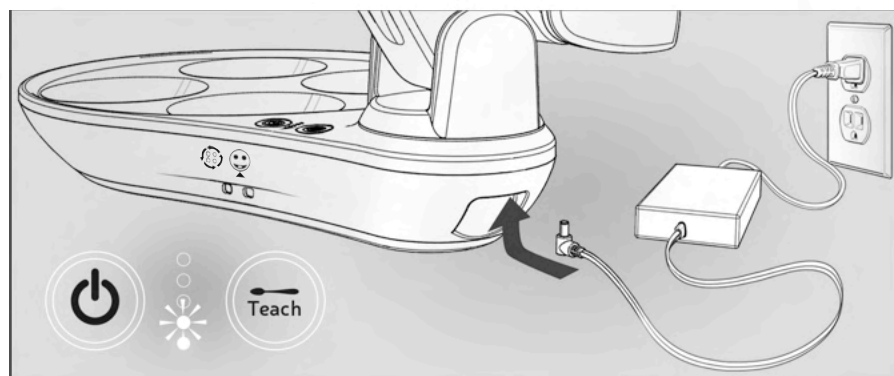

Connect the Charging Cable to Obi's Charge Port and an electrical outlet. All Battery Indicator Lights will illuminate when the device is fully charged.



If you experience an issue with Obi, Email: [service@meetObi.com](mailto:service@meetObi.com) or Call: 1-844-HelpObi.

### SERVICE